Application :

The M15AE / M17AE;MA17BH / M19AI is GVision's medical monitor for hospital applications. Designed for medical use, GVISION monitors with optional touchscreen can be easily integrated into a multitude of solutions: healthcare applications such as nursing stations, computer-aided therapy, electronic medical record keeping, paperless charting, patient self-check-in / registration or point-of-information installations.

Table of Contents

1. Cautions and Warnings	2
2. VESA mounting installation	6
2-1. 15" LCD Monitor VESA Mounting	6
2-2. 17"/19" LCD Monitor VESA Mounting	7
3. FCC Statement	8
4. WEEE Statement	9
5. Japan RoHS Declaration	10
6. Introduction	12
7. Monitor Controls	13
7-1. Key Function	13
7-2. Quick Adjustment Function	13
7-3. Display Angle	14
7-4. Stand setting up	14
7-5. I/O Port	15
7-6. OSD Control options	15
8. Screen Adjustment	16
9. Touch installation	18
10. Power Management	19
11. Trouble shooting	20
12. Display Modes	21
13. Specification	22

1. Cautions and Warnings

Medical equipment as to electric shock, fire and mechanical hazards only in accordance with ANSI/AAMI ES 60601-1 (2005)+ AMD 1 (2012) AND CAN/CSA-C22.2 No. 60601-1:14

Model: M15AE / M17AE; MA17BH / M19AI

Symbol Description

Operating Instructions!
Consult Accompanying Document
Mode d'emploi!
Consultez le document d'accompagnement

Caution ! – This symbol alerts you to important operating considerations or a potential operating condition that could damage equipment. Refer to user's manual or operation's manual for precautionary instructions.

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Class I.

No applied parts.

Protection against harmful ingress of water : Ordinary Equipment (IPX0) – No protection.

Not suitable for use in the presence of a flammable anesthetic's or oxygen.

Mode of operation : Continuous.

Warning

- To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.
- Do not modify this equipment without authorization of the manufacturer.
- Permitted with service personnel to install and repair only.

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Precautions

- Read all of these instructions and save them for later use.
- Follow all warnings and instructions on the product.
- Product
 - Do not cover or block the vent holes in the case.
 - Do not insert sharp objects or spill liquid into the LCD monitor through cabinet slots. They may cause accident fire, electric shock or failure.
 - Disconnect the power plug from the AC outlet if you will not use it for an indefinite period of time.
 - Do not attempt to service this product yourself, as opening or removing covers may expose you to dangerous voltage points or other risks.
 - Do not apply pressure to the screen. The LCD is very delicate.
 - Do not touch the LCD panel or Touch Screen surface with sharp or hard objects.
- Removed power cord if fault happened.
- Patients are not accessible to touch the panel when the monitor is turning on (or turn off)
- Disconnect device of the EQUIPMENT is plug. For pluggable equipment, the power outlet must be installed near the equipment and must be easily accessible.
- Power
 - Use the type of power indicated on the marking label.
- Power Adapter : Only use an adapter designed for the LCD monitor, model M15AE, M17AE.

Adaptor type : Sinpro , MPU64-105.

- Rating : Input 100-240Vac , 47-63Hz , 1.62A~0.72A ; Output 12Vdc, 5.25A, 63W
- Power Rating : Input 100-240Vac , 50-60Hz , 1.6~0.7A for M19AI

Manufacturer : Gvision Incorporated

4F., No.87, Lide St., Zhonghe Dist., New Taipei City 235, Taiwan

Tel : +886-2-8226-1110 ; Fax : +886-2-8226-1112.

IMPORTANT – Use of another type of adapter will result in malfunction and/or danger.

- The device could be sent back to manufacturer for recycling or proper disposal after their useful lives. Alternatively the device shall be disposed in accordance with national laws after their useful lives
 - ◆ If you will not use it for an indefinite period time.
 - ◆ When the power cord or plug is damaged or frayed.
 - ◆ If the product does not operate normally when the operating instructions are followed. Adjust only those controls that are covered by the operating instructions. Improper adjustment of other controls may result in damage and will often require extensive work by a qualified technician to restore the product to normal operation.
 - ◆ If the product has been dropped or the cabinet has been damaged.
 - ◆ If the product exhibits a distinct change in performance, indicating a need for service.
- Power and extension cords
 - Do not allow anything to rest on the power cord.
 - Do not locate this product where persons will walk on the cord.
 - Use the proper power cord with correct attachment plug type. If the power source is 120 V AC, use a power cord that has UL and C-UL approvals. If the power source is a 240 V AC supply, use the tandem (T blade) type attachment plug with ground conductor power

cord that meets the respective European country's safety regulations, such as VDE for Germany.

- Do not overload wall outlets or power cords. Ensure that the total of all units plugged into the wall outlet does not exceed 10 amperes.
- Ensure that the total ampere ratings on all units plugged into the extension cord is not above the cord's rating.
- If the power supply cord, which came with your monitor, is to be connected to the PC instead of the wall outlet, this equipment is to be used with UL/TÜV approved computers with receptacle rated 240V AC, 2.0A (minimum).
- Environment
 - Place the monitor on a flat and leveled surface.
 - Place the monitor in a well-ventilated place.
 - Keep the monitor away from: overly hot, cold or humid places, places directly under sunlight, dusty surroundings, equipment that generate strong magnetic fields.
 - Not suitable for use in the presence of a flammable substance.
 - To prevent fire or shock hazard, never expose the monitor to rain or moisture.
- Option Mounting Methods: Either Base mounting or VESA mounting can be used.

"Accessory equipment connected to the analog and digital interfaces must be certified according to the respective IEC standards (e.g. IEC 60950 for data processing equipment and IEC 60601-1 for medical equipment). Furthermore all configurations shall comply with the valid version of the system standard IEC 60601-1-1. Everybody who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore responsible that the system complies with the requirements of the valid version of the system standard IEC 60601-1-1. If in doubt, consult the technical service department or your local representative. " (MDD: 13.6.c, IEC 60601-1: 6.8.2.c, 19.2.b, 19.2.c)

External Equipment

External equipment intended for connection to signal input/output or other connectors, shall comply with relevant UL/IEC standard (e.g. UL 1950 for IT equipment and UL 60601-1 / IEC 60601 series for systems – shall comply with the standard IEC 60601-1-1, Safety requirements for medical electrical systems. Equipment not complying with IEC60601-1-1 shall be kept outside the patient environment, as defined in the standard.

2. VESA mounting installation

2-1. 15" LCD Monitor VESA Mounting

Fix UL listed wall mount bracket on the back of the monitor with screws, the specification of screw (ref. image 1) , to prevent electric damage to ensure user's safety. The dimension of interface pad of wall mount bracket has to follow VESA standard (ref. Image 2)

Installation: Please aim UL listed wall mount bracket at the 4 screw holes on the back of monitor and screw tightly (ref. Image 3) .

Please refer to user's manual of UL listed wall mount bracket for wall mount installation.

SCREW M4L8 ,EGW,PAN(+),NI

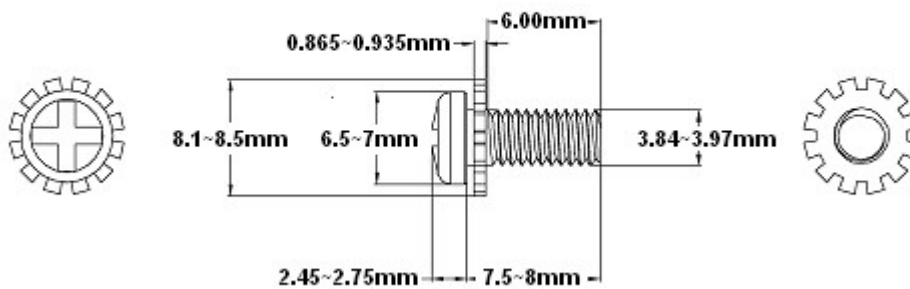
Image 1

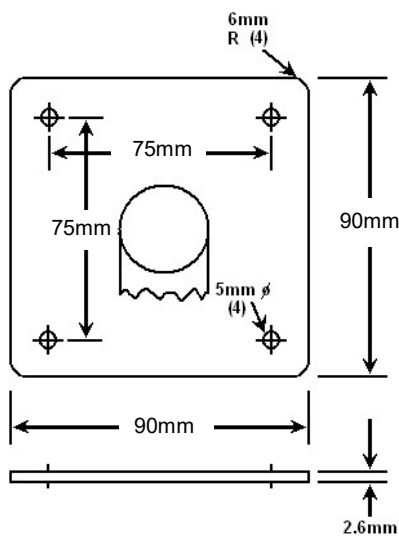
Image 2

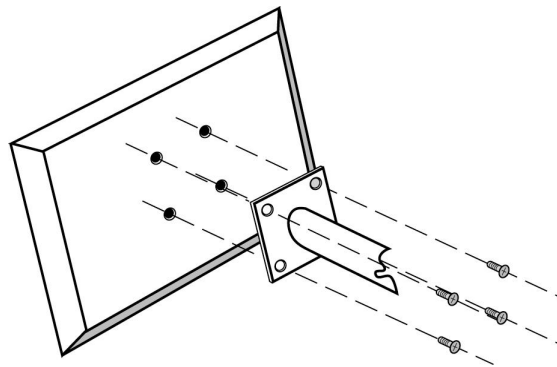
Image 3

VESA[®] Standard Interface Pad Configurations

2-2. 17" / 19" LCD Monitor VESA Mounting

Fix UL listed wall mount bracket on the back of the monitor with screws, the specification of screw (ref. image 1) , to prevent electric damage to ensure user's safety. The dimension of interface pad of wall mount bracket has to follow VESA standard (ref. Image 2)

Installation: Please aim UL listed wall mount bracket at the 4 screw holes on the back of monitor and screw tightly (ref. Image 3) .

Please refer to user's manual of UL listed wall mount bracket for wall mount installation.

SCREW M4L10,EGW,PAN(+),NI


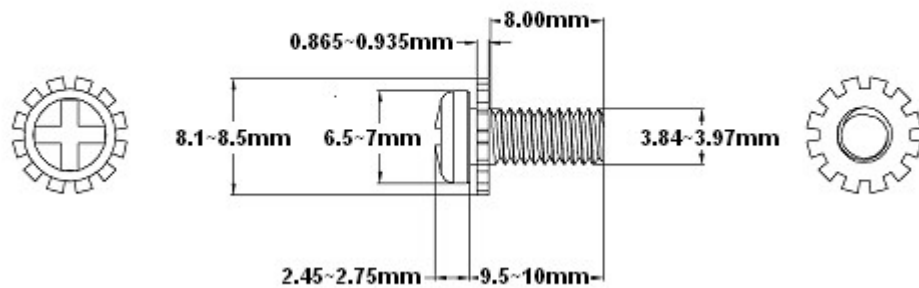
Image 1

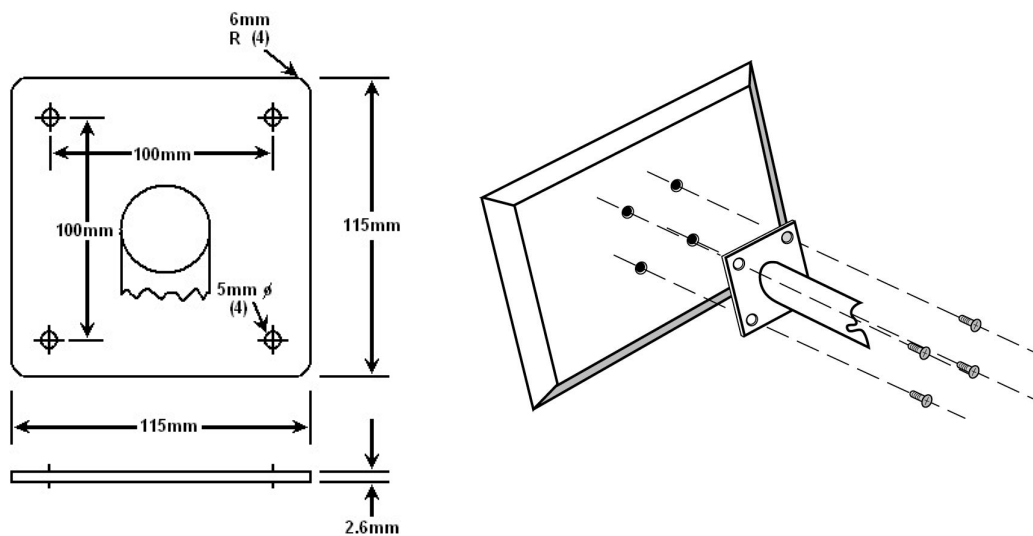VESA[®] Standard Interface Pad Configurations

Image 2

Image 3

3. Federal Communication Commission Radio Frequency International Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more the following measures:

- Reorient / Relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Notice:

The use of non-shielded interface with this equipment is prohibited.

If necessary, the user should consult the dealer or an experienced radio / television technician for additional suggestions. The user may find the following booklet prepared by the Federal Communications Commission helpful. This booklet is available from the U.S. Government Printing Office Washington DC, 20402, Stock No. 004-000-00345-4.

The manufacture is not responsible for any radio or TV interference caused by unauthorized modification of this equipment. It is the responsibility of the user to correct such interference.

Caution: To meet FCC limits for a Class B computing device, a shielded signal cable should be used.

Canadian Department Of Communications Compliance Statement

This equipment does not exceed Class B limits per radio noise emissions for a digital apparatus, set out in the Radio Interference Regulation of the Canadian Department of Communications. Operation in a residential area may cause unacceptable interference to radio and TV reception requiring the owner or operator to take whatever steps necessary to correct the interference.

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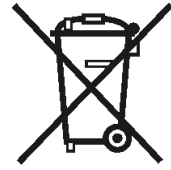
4. WEEE Statement

Important Information for Recycling

This warning sign of trash bin indicates that the objects are not allowed to be disposed as general refuse. Please follow the local Act and carry out further disposal with refuse recycling and collecting system in your Country or District. For more information on recycling, please contact locally authorized recycling centre or place of purchase.

Your recycling behaviour will bring better future for the Earth and create living environment with excellent quality for the descendant.

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6. Introduction

◆ Product Features

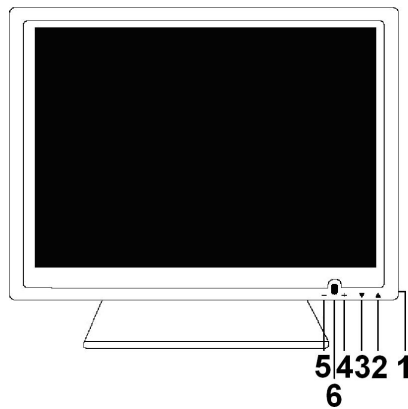
- High contrast color TFT LCD display support resolution up to XGA (15" model) and SXGA 1280 x 1024 (17" & 19" model).
- Compatible with IBM VGA,DVI, VESA standards.
- Imbedded audio system provides sound performance.
- Power management system conforms to VESA DPMS standard.
- Universal power adapter.
- Support DDC1/2B for Plug & Play compatibility.
- Advanced OSD control for picture quality adjustment.
- Support VESA standard mounting holes for wall mount or ARM mounting application.
- Touch screen system (Optional)
- Digital interface (Optional)
- The aluminum stand base tilt down -5°~ tilt up +90°

◆ Package Contents


- 15" / 17" / 19" TFT liquid crystal monitor
- D-sub 15 pin signal cable for analog input
- DVI 24 pin signal cable for digital input (Optional)
- RS232 / USB cable for touch screen function (option).
- RCA / S-Video signal cable for video-in function (option).
- Audio cable (Optional)
- Power adapter / AC power cord (Optional)
- Touch screen CD driver (Optional)
- 4 pieces M4x10mm washer screws (Optional)
- User's manual

7. Monitor Controls

7-1. Key Function

△BX / BH Serial

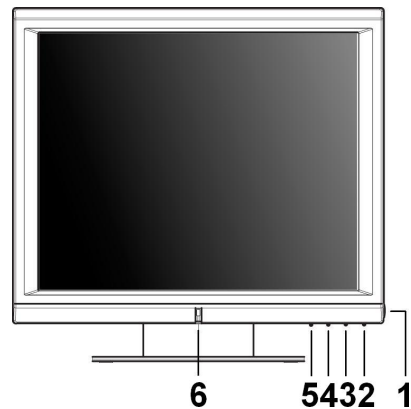△VX / VH Serial

1. **Power Switch** Pressing this button turns the display system power **on** or **off**.
2. **Menu/Select “▲”** Pressing this button opens the OSD menu on the screen, and is used to select (“Up” direction) the OSD control options on the screen.
3. **Select “▼”** This button is used to select (“Down” direction) the OSD control options on the screen.
4. **“+”** This button is used to increase the value of the selected OSD control option.
5. **“-”** This button is used to decrease the value of the selected OSD control option.
6. **Power LED on or off.**

7-2. Quick Adjustment Function

When the OSD Menu is not active, the following functions are available for the quick adjustment.

- Pressing “▲” button once pops up the OSD Function Main Menu .
- Pressing “▼” button once pops up the small Brightness icon. Use “+” or “-” button to adjust the Brightness level.
- Pressing “+” button once pops up the small Volume icon. Use “+” or “-” button to adjust the Volume level.
- Pressing “-” button once pops up the small Input Select icon. Press “-” button again to switch to the digital interface. (Only for the model with digital input)
- Pressing “+” button once and then press both “+” and “-” buttons at one time, the audio will be mute .Pressing “+” button again, the audio come back .

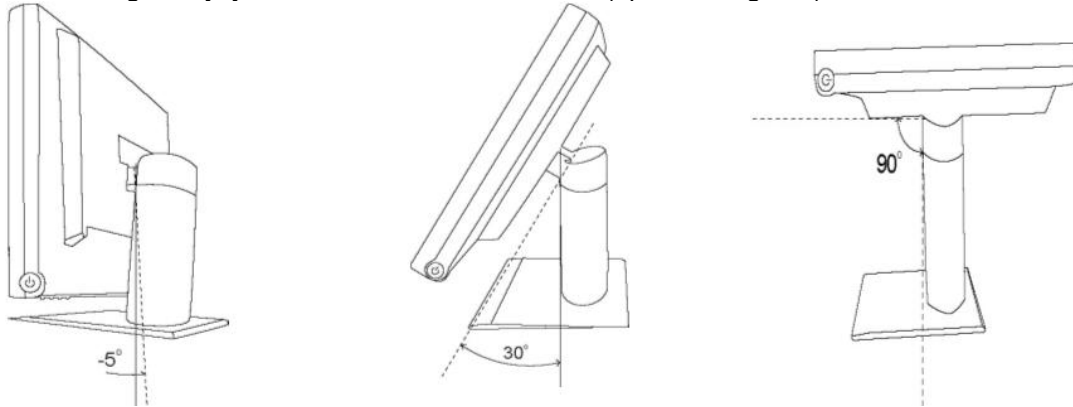
Note:

Key lock function : Press “▼” button and then press Power switch together to freeze all function keys. Press “▼” button and then press Power switch together again to unlock the keys.

7-3. Display Angle

Tilting


For viewing clarity, you can tilt the LCD backward . (up to 90 degrees).

CAUTION : In order to protect the LCD, be sure to hold the base whenever you adjust it.


7-4. Stand setting up

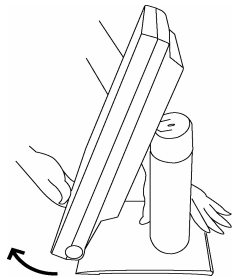Step 1


Hold the stand tight and pull monitor out upward.

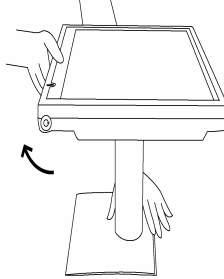
Step 2

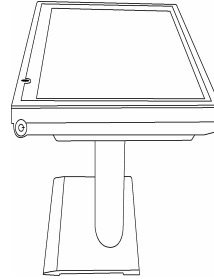Step 3

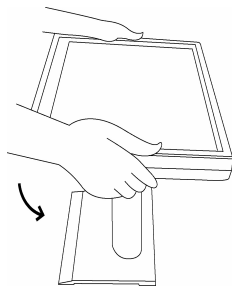
Step 4


Continuing step , then out down the entire monitor downward.

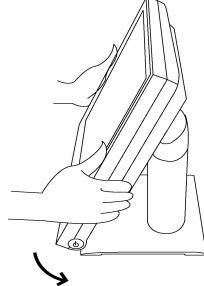Step 5

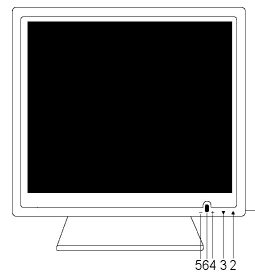Step 6

Place the monitor in an upright position.

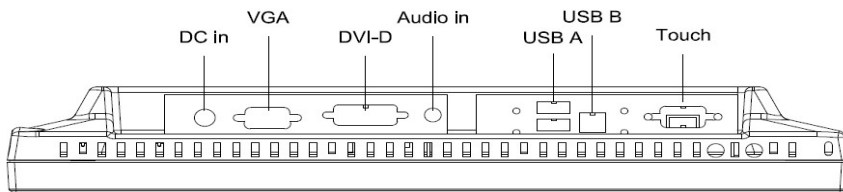
WARNING : DO NOT USE falls short of the specifications of screw for VESA mounting. Strongly recommend use the 4 screws we provided.

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
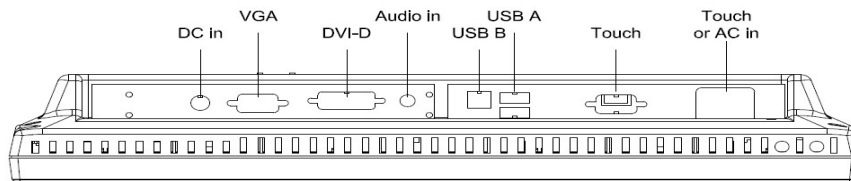
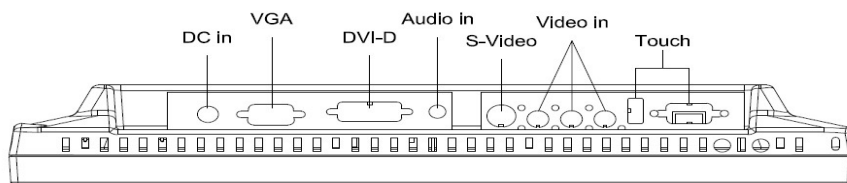
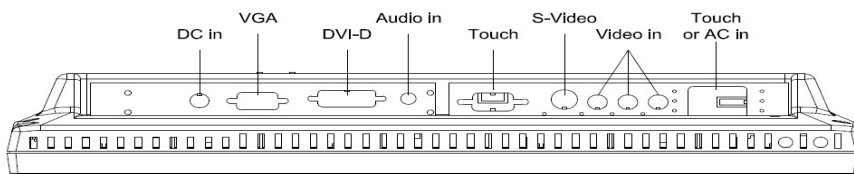
7-6. OSD Control options

Pressing " - " button once opens the small **Input Select** icon.

PC ANALOG
PC DIGITAL (Optional)
VIDEO (Optional)
S-VIDEO (Optional)
EXIT

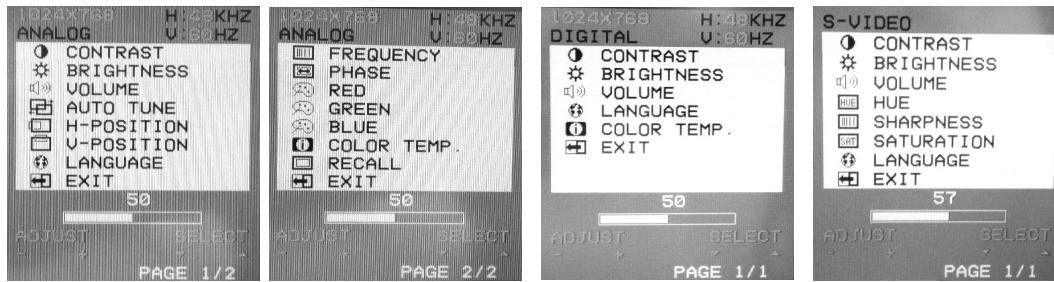8. Screen Adjustment

PC Analog Mode

PC Digital Mode

S-Video/Video Mode

- **CONTRAST**
Adjust the contrast using “+” and “-” buttons.
- **BRIGHTNESS**
Adjust the brightness using “+” and “-” buttons.
- **VOLUME**
Adjust the speaker volume using “+” and “-” buttons.
- **AUTO TUNE**
Press “+” or “-” buttons to make LCD monitor adjust the related parameters automatically for optimal display status.
- **H - POSITION**
Pressing “+” moves the display image to the right; Pressing “-” moves the display image to the left.
- **V - POSITION**
Pressing “+” moves the display image upward; Pressing “-” moves the display image downward.
- **LANGUAGE**
This item is used to select OSD languages.
- **FREQUENCY**
This item is used to decrease the vertical interference.
 1. Select one display pattern from your computer.
 2. Use “+” and “-” buttons on the LCD monitor to decrease the vertical interference to get the optimal display.
- **PHASE**
This item is used to decrease the horizontal interference.
 1. Select one display pattern from your computer.
 2. Use “+” and “-” buttons on the LCD monitor to decrease the horizontal interference to get the optimal display.
- **RED / GREEN / BLUE**
Adjust the value of color gain (Red, Green, Blue) using “+” and “-” buttons.

- **COLOR TEMP.**
This section is for “Color Temperature” adjustment. You can select the Warm, Cool, or Normal in this section, and then press “+” and “-” buttons to set up. The color temperature will be recalled as factory preset value.
- **RECALL**
Press “+” or “-” buttons to recall factory preset values.
- **HUE** (For the model with Video input only)
Adjust the video image hue using + or - buttons to set up.
- **SHARPNESS** (For the model with Video input only)
Adjust the sharpness using + or - buttons to set up.
- **SATURATION** (For the model with Video input only)
Adjust the video image color saturation using + or - buttons to set up.

9. Touch installation

◆ **Installing Touch Screen Controller Driver**

To install the driver of the touch screen controller to your computer, please follow the steps below:

1. Insert the CD-ROM in your computer's CD-ROM drive.
2. If autorun is enabled, the setup program appears automatically. If autorun is disabled, run setup.exe from the root directory of CD.
3. Follow the directions on the screen to complete the installation.
4. After finishing the installing, restart the computer as instructed.

◆ **Uninstalling Touch Screen Controller Driver**

To uninstall the driver of the touch screen controller from your computer, please follow the steps below:

1. Click the Start button, click Control Panel, and then click Add/Remove Programs.
2. Select Touch Screen Controller Driver from the list of uninstallable programs on the Install/Uninstall tab.
3. Click the Add/Remove button then follow the directions on your screen.
4. After finishing the uninstalling, restart the computer as instructed.

10. Power Management

General Power Saving Mode

Mode	Power LED Color	Power Consumption			
		Monitor	Monitor+ Touch	Monitor+Video +Touch	48W (max.)
On	Green				
Stand by	Orange	2W	2W	4W	
Suspend	Orange	2W	2W	4W	
Off	Orange	2W	2W	4W	

* We recommend to switch off the monitor when it is not use for quite a long time.

11. Trouble shooting

The monitor does not respond after you turn on the system.

- Check if the **Power Switch** of monitor is turned **on** .
- Turn off the power and check the monitor's power cord and signal cable for proper connection.

The characters on the screen are dim.

- Refer to the Controls and Adjustments section to adjust the brightness.

The screen is blank.

- During operation, the monitor screen may automatically turn off as a result of Power Saving feature. Press any key to see if the screen comes back.
- Refer to the Controls and Adjustments section to adjust the brightness.

The screen flashes when it's initialized.

- Turn off the monitor and turn it on again.

◆ **Cleaning**

- Gently wipe the screen with a clean camel hair lens brush, or a soft, clean, lint-free cloth. This removes dust and other particles that can scratch the screen.
- Do not apply pressure to the screen surface when wiping it clean.
- Do not pour or spray any liquid directly onto the screen or case of the LCD monitor. Chemical cleaners have been reported to damage the screen or case of the LCD monitor.
- **No scheduled maintenance or calibration activities required.**

WARNING : Do not disassemble the monitor. Contact your dealer if needed.

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13. Specification


LCD Display		15" / 17" / 19" TFT Active Matrix Panel
Panel Display Size	15"	304.1(H) x 228.1 (V) mm
	17"	337.9 (H)×270.3 (V) mm
	19"	376.3(H)×301.05(V) mm
Panel Pixel Pitch	15"	0.297(H) x 0.297 (V) mm
	17"	0.264(H) x 0.264 (V) mm
	19"	0.294(H) x 0.294(V) mm
Panel Max. Resolution	15"	XGA 1024 x 768
	17"	SXGA 1280 x 1024
	19"	SXGA 1280 x 1024
Panel Contrast Ratio	15"	700 : 1(typical)
	17"	800 : 1(typical)
	19"	800 : 1(typical)
Panel Brightness	15"	250 Cd/m ² (typical)
	17"	300 Cd/m ² (typical)
	19"	300 Cd/m ² (typical)
Panel Response Time	15"	8 ms (typical)
	17"	5 ms (typical)
	19"	5 ms (typical)
Panel Display Color	15"	16.2M (typical)
	17"	16.7M (typical)
	19"	16.7M (typical)
Panel Viewing Angle	15"	140° (L/R), 125° (U/D) (typical)
	17"	160° (L/R), 160° (U/D) (typical)
	19"	160° (L/R), 160° (U/D) (typical)
Input Signal	Video	R.G.B. Analog 0.7V _{peak-peak}
	Sync	TTL Positive or Negative
Signal Connector	Analog	15 Pin D-Sub
	Digital (Option)	24 -pin DVI-D
Plug & Play		DDC1 / 2B Compatible
Audio System		1W / Per Channel
USB HUB (Option)		1 Upstream Port, 2 Downstream Ports (Providing touch interface for USB-A and USB-B type)

Touch Screen (Option)		USB Type or RS232 or Combo
Video system (Option)		RCA, S-Video
Power	15" Power Adapter	Input AC 100~240Vac, 47~63Hz, 1.62A~0.72A . Output DC 12V / 5.25A .
	17" Power Adapter	Input AC 100~240Vac, 47~63Hz, 1.62A~0.72A . Output DC 12V / 5.25A .
	19" Internal Power	Universal 100 ~ 240Vac, 50~60Hz, 1.6~0.7A . AC input
Fuse Rating	19" Internal Power	T2 AL , 250Vac (provided on both Fuse Holder)
Operating Conditions	Temperature	0°C ~ 40°C (32°F ~ 104°F) [Note]
	Humidity	20% ~ 80% (no condensation) [Note]
	Altitude	To 10,000 Feet
	Pressure	697~1060 hpa
Storage / Transport Conditions	Temperature	-20°C ~ 60°C (-4°F ~ 140°F) [Note]
	Humidity	10% ~ 90% (no condensation) [Note]
	Pressure	187~1060 hpa
Stand		Tilt down : -5° ~ Tilt up : +90°
Monitor Dimensions (W x H x D)	15"	351 x 314.5 x 140 mm
	17"	392 x 381.5 x 180 mm
	19"	438 x 417.5 x 180 mm
Carton Dimensions (W x H x D)	15"	438 x 460 x 214 mm
	17"	452 x 461 x 228 mm
	19"	530 x 500 x 230 mm
Monitor and Stand Weight (Net)	15"	4.5 Kg
	17"	5 Kg
	19"	7 Kg
Total Weight	15"	5.2 Kg
	17"	7 Kg
	19"	9.1 Kg
Certifications		UL , C-UL , FCC , CE , TUV
Compliance with IEC60601-1-2 EMC requirement and UL 60601-1		

- Specifications are subject to change without notice.

[Note]

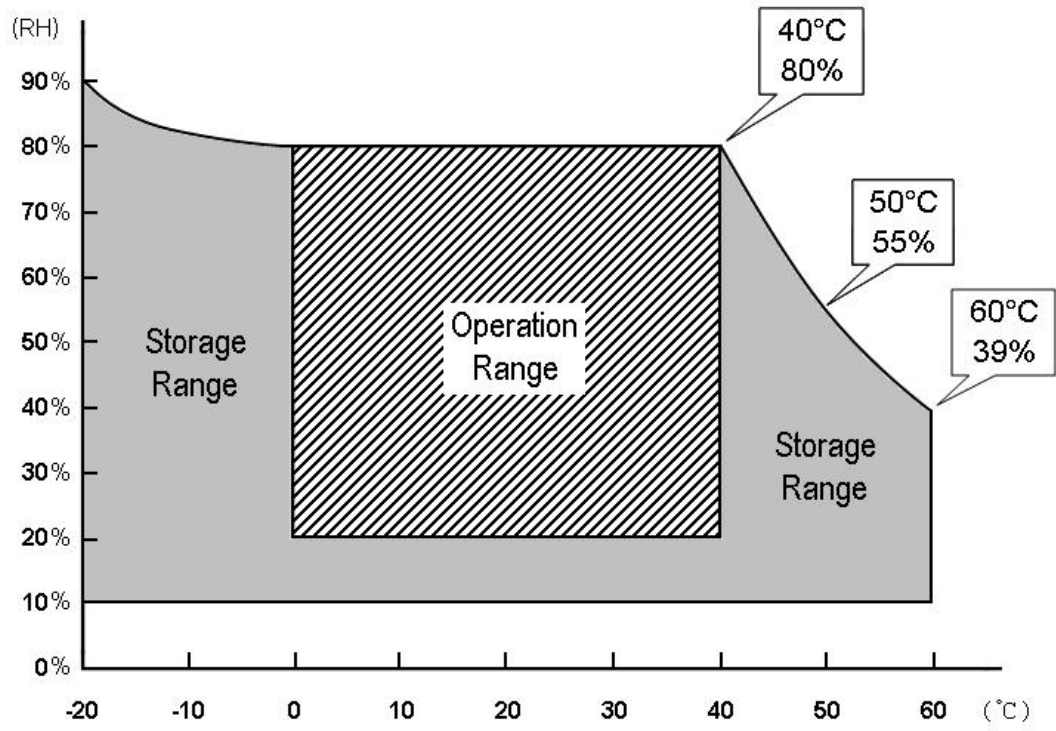For quality performance. Please refer the graph below for the corresponding of temperature and humidity.

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